POLSKI

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StethoMe®. Smart Stethoscope.

## Spis treści

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StethoMe®. Smart Stethoscope.

## Table of contents

<b>1. Introduction</b> .....	<b>5</b>
1.1 About the manual.....	5
1.2. Index of revisions.....	5
1.3. Symbols and markings used in the manual.....	6
<b>2. Characteristics</b> .....	<b>6</b>
2.1 Intended use of the medical device.....	6
2.2 Contraindications.....	6
2.3 Potential adverse effects and medical device safety.....	6
2.4 Precautions and warnings.....	6
2.5 Additional information.....	9
2.6. Medical device classification.....	9
2.7. Markings.....	10
2.8. Label.....	10
<b>3. Equipment and structure</b> .....	<b>10</b>

3.1. Equipment.....	10
3.2. Structure.....	11
3.3. Connectors.....	12

<b>4. Preparations for use</b> .....	<b>13</b>
4.1. Dedicated application.....	13
4.2. StethoMe stethoscope.....	13

<b>5. Activities during operation</b> .....	<b>14</b>
5.1. Patient auscultation mode using a mobile application without connection to the StethoMe AI medical device.....	15
5.2. Patient auscultation mode using a mobile application with connection to the StethoMe AI medical device.....	16
5.3. Patient auscultation mode without using a mobile application.....	17
5.4. Turning off the stethoscope.....	18

<b>6. Maintenance and cleaning</b> .....	<b>18</b>
--	-----------

<b>7. Troubleshooting</b> .....	<b>18</b>
---------------------------------	-----------

<b>8. Technical specification</b> .....	<b>18</b>
8.1. Electromagnetic compatibility and compliance with standards .....	19
8.2. Storage, transport and operation conditions .....	19
<b>9. Maintenance service</b> .....	<b>20</b>
<b>10. Manufacturer</b> .....	<b>20</b>

# 1. Introduction

## 1.1 About the manual

This manual is a part of a medical device - the StethoMe Smart Stethoscope.

StethoMe Sp. z o.o. bears no liability and provides no guarantee with regard to damage (including indirect damage) arising due to this user manual not being complied with.

- Prior to using the medical device it is required to read this user manual carefully.
- This user manual should be stored in a safe place for the whole duration of medical device use.
- It should be provided to every subsequent owner or user of the medical device.
- It should be updated on the basis of every supplementation received from the medical device manufacturer.

The aim of this manual is to describe a medical device – the StethoMe Smart Stethoscope – taking into account the following in particular:

- Medical device description
- Safety of use
- Maintenance
- Troubleshooting
- Servicing

## 1.2. Index of revisions

This user manual is valid for the below specified version of the StethoMe Smart Stethoscope.

Medical device version: 2.0.0

Status: 2019/07

## 1.3. Symbols and markings used in the manual

Warnings and safety measures that have to be followed in order to avoid personal injury!

Warnings and safety measures that have to be followed in order to avoid damage to property!

Tips/additional information.

## 2. Characteristics

### 2.1 Intended use of the medical device

The StethoMe medical device enables recording the respiratory sounds or heart sounds by a physician, other medical personnel or a person without medical education, and remote transmission of such records. StethoMe can be combined with

the StethoMe AI medical device in order for the signals to be analysed by artificial intelligence algorithms.

**CAUTION! Any other application of the device is considered inconsistent with the intended use and has to be excluded.**

### 2.2 Contraindications

Do not perform auscultation if there are any wounds or abrasions within the examined point.

### 2.3 Potential adverse effects and medical device safety

This medical device is completely safe for people, provided that it is used according to the user manual.

### 2.4 Precautions and warnings

- This medical device is an electronic stethoscope. It is certified and should be used solely for recording the respiratory system sounds or heart sounds and transmitting them remotely. The medical device does not analyse the sounds recorded. The medical device may but does not have to be used in combination with the StethoMe AI medical device, enabling analysis of the recordings transmitted, within the ranges specified, for presence of certain abnormal additional sounds indicated in the user manual for this medical device.
- Using the medical device is not a substitute for visiting a physician and may not delay obtaining medical assistance in sudden pathological conditions.
- The medical device can be operated by adults only.
- Do not perform auscultation with the medical device if there are any wounds or abrasions within the examined point.
- If the medical device user has a pacemaker implanted, the admissibility of using the medical device should be consulted with a physician.
- We underline that the medical device is sensitive to external factors – noises or murmurs may have a negative impact on the quality of recordings.
- Prior to commencing examination it is necessary to become familiarised in detail with the medical device user manual available at the address: [www.StethoMe.com/manuals](http://www.StethoMe.com/manuals).
- This medical device is not intended for diagnostics in emergency conditions! Remember that if you suspect any hazard to life or health (e.g. trouble breathing, impaired consciousness, significantly increased respiratory rate, panic), do not use the medical device! In such a situation, consult a physician or report the problem using the 112 emergency line.
- In case of any questions related to how the medical device should be used (e.g. regarding the point where the medical device should be put to the body), please contact a physician.
- The recordings made using the medical device may be transmitted only via the StethoMe application or applications of partners authorised by the medical device manufacturer. The recordings made using any applications other than those specified above are not reliable, which may lead to their content being interpreted incorrectly by a physician and, in consequence, to incorrect diagnosis. The list of authorised applications is available at the website: [www.StethoMe.com/apps](http://www.StethoMe.com/apps).
- The application has been tested only on the mobile phone models indicated. Although it is possible that the application may work on other mobile phone models, the manufacturer does not guarantee correct functioning in such cases. Using the application with any mobile phone other than

those listed in this manual should be considered inconsistent with the intended use. The list of mobile phone models supported is available at the website: [www.StethoMe.com/devices](http://www.StethoMe.com/devices).

- In case of any questions regarding the StethoMe medical device, please contact us at the address: [support@stethome.com](mailto:support@stethome.com) or contact the medical personnel attending to the patient.
- The medical device manufacturer bears no liability for the medical device being used in a manner that does not comply with the user manual or the intended use.
- The medical device includes small elements that can be ingested or aspirated: protect the medical device and its accessories against free access of children.
- The power supply cable may constitute a strangulation hazard: any examination of children should be conducted under supervision.
- Only the special USB [type A] - 3.5 mm mini jack [4-pin] cable provided by the manufacturer and the charger provided by the manufacturer should be used for charging the device.
- The USB [type A] - 3.5 mm mini jack [4-pin] cable provided by the manufacturer and the charger provided by the manufacturer features electronic ele-

ments inside the USB type A plug and constitutes a part of the complete set.

- Do not perform examinations or put the device to the body while the device is being charged.
- When charging, do not place the stethoscope in a manner that would make disconnecting it from the power socket difficult.
- Do not attempt to replace the battery on your own.
- Prior to commencing examination it is **ABSOLUTELY** necessary to disconnect the charging cable from the device!

- The device should be used in compliance with the intended use.
- The stethoscope should be used in ambient temperature from 15 to 40°C.
- Do not expose the stethoscope to temperatures greater than 50°C or lower than 0°C.
- Do not use the stethoscope when the relative humidity exceeds 85%.
- Do not expose the device to impact of any chemical substances, sunlight, or high temperature.
- It is always necessary to adhere to the fundamental principles of safety, particularly when the medical device is used near children.
- Prior to using the stethoscope it is necessary to make sure that the stetho-

scope has no visible damage.

- The stethoscope should not be opened or tampered with.
- The stethoscope should not be immersed in water or other liquids.
- Protect the stethoscope against shocks and falls.
- The stethoscope should be stored in closed original packaging, in a place out of sight and reach of children.
- In case of noticing any irregularities in functioning of the stethoscope or any damage to it, stop using it immediately and contact the manufacturer.

## 2.5 Additional information

- In order for the medical device to offer the whole range of its functions, an Internet connection is required.
- This medical device is intended for repeated use.
- It is intended for both home and professional use.
- The stethoscope should be stored in a clean and dry place.
- The stethoscope is not sterile and must not be sterilised.
- The stethoscope should be charged only by means of the charger included in the set supplied.

- While charging the device, all its functions, including data transmission, are blocked.
- The stethoscope does not include any parts that can be replaced by the user on their own.
- This manual should be retained so that it can be consulted again in the future if necessary.

## 2.6. Medical device classification

1. The StethoMe medical device has been classified according to Annex IX to the Directive of the Council 93/42/EEC as belonging to class IIa and is subject to rule 10.
2. The stethoscope is powered internally: lithium-ion battery, not replaceable.
3. Type of application parts applied: BF.
4. Protection class: IP22.
5. Does not require sterilisation.
6. Mode of functioning: continuous.

## 2.7. Markings

-  Manufacturer
-  Direct current
-  **CAUTION! IMPORTANT!**  
Please read the user manual
-  Serial number
-  Protect against humidity
-  Caution - fragile  
Easily breakable load
- IP22** Water resistance
-  Application part of the BF type

 **2274** The device meets the requirements of the 93/42/EEC directive and harmonised standards

 Disposal of waste electric and electronic equipment – the medical device should be disposed in accordance with national regulations regarding waste electric and electronic equipment

 Please familiarise yourself carefully with the user manual prior to using the medical device.

## 2.8. Label

 **StethoMe**  StethoMe sp. z o.o.  
www.StethoMe.com

Smart Stethoscope 61-633 Poznań, Poland  
ul. Winogrody 18A

 **2274**

5V DC  250mA  
**IP22**

 **SN** 00000000

## 3. Equipment and structure

### 3.1. Equipment

The set includes:

1. StethoMe Smart Stethoscope [Fig. 1],
2. special USB - mini jack 3.5 mm cable [Fig. 2a],
3. charger [Fig. 2b],
4. user manual,
5. warranty card,
6. quick start guide.

Fig. 1

Fig. 2a

Fig. 2b

## 3.2. Structure

The main elements of the stethoscope are presented on Fig. 3a and Fig. 3b.

Fig. 3a

Fig. 3b

1. LCD display.
2. 3.5 mm mini jack socket - used for connecting headphones and for charging.
3. Membrane.
4. Ventilation openings.

**IMPORTANT!** In order to take advantage of all features of StethoMe, it is required to have Internet access. Prior to conducting an examination, it is necessary to download the dedicated certified application from Google Play or App Store.

The table below includes the description of the most important symbols displayed on the stethoscope screen:

	StethoMe® logo
	Stethoscope is connected to the StethoMe application

	Stethoscope battery charge level
	Headphones are connected to the stethoscope
	Stethoscope is being charged
	Heart auscultation filter active
	Lung auscultation filter active
	Recording finished
	Recording interrupted [red symbol]

	Remain silent
	It is too loud or stethoscope is not put to the body not stable [red symbol]

### 3.3. Connectors

The StethoMe stethoscope is fitted with a single 3.5 mm mini jack socket intended for:

- connecting headphones in order to listen to the sounds recorded,
- connecting the USB-3.5 mm mini jack cable, included in the set supplied, for charging the device [the device is inactive while being charged].

The description of 3.5 mm jack plug sections is included on Fig. 4.

Fig. 4. 1 - left headphone channel [inactive while charging], 2 - right headphone channel [inactive while charging], 3 - GND [ground], 4 - ground [headphones] / +5V [charging].

## 4. Preparations for use

### 4.1. Dedicated application

Prior to using the stethoscope, it is necessary to download the dedicated certified application from Google Play or AppStore and install it, accordingly, on a mobile device with Android 5.0 system or newer, or a mobile device with iOS 11 system or newer.

The list of mobile phone models supported is available at the website: [www.StethoMe.com/devices](http://www.StethoMe.com/devices)

### 4.2. StethoMe stethoscope

1. When using the stethoscope for the first time, and after it was turned off, it is necessary to switch the device on using the cable included in the set [fig. 2a] by inserting and removing the 3.5 mm jack plug into the stethoscope's jack socket [Fig. 5].

**IMPORTANT!** If the battery of the device is charged [the battery status icon is located in the upper right corner of the screen], the cable can be disconnected after the start screen [with StethoMe logo] appears.


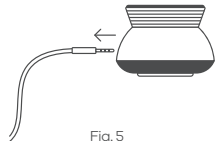
Fig. 5

2. In order to charge the battery, the device has to be connected to the charger included in the set using the included cable, and then left connected to a mains socket until it is fully charged [the battery icon is filled].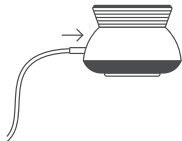

**IMPORTANT!** When the device is being charged, all its function are blocked.

When the stethoscope is charged, switched on and in standby mode, it is not necessary to connect it to the USB port. It is enough to shake it gently in vertical axis until the start

screen [StethoMe logo, Fig. 6] appears.

3. Communication between the phone and the stethoscope is possible after activating the Bluetooth and Location functions in the phone.

4. It is necessary to launch the dedicated certified application, which will connect to the stethoscope after the type of examination is selected. Depending on the phone and the quality of wireless signal, the connection time may differ - it usually takes about 5 seconds. During that time, the icon of connection with the phone will appear on the stethoscope, indicating readiness to work:

**IMPORTANT!** When launching, the application checks the version of stethoscope and its soft-

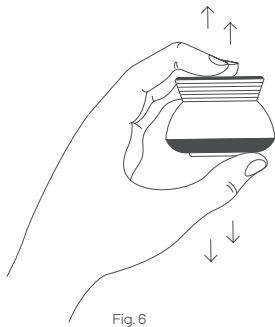
Fig. 6

ware. If necessary, the application will carry out an automatic update process. A software update may take about 1 minute and the user will be notified of its progress on a current basis. When the update is in progress neither the phone nor the stethoscope should be used, as this may damage the device.

## 5. Activities during operation

The StethoMe stethoscope can be used in one of the three modes listed below:

- with a mobile application without connection to the StethoMe AI medical device
- with a mobile application and with connection to the StethoMe AI medical device
- without a mobile application

**CAUTION!** When recording, make sure that the jack socket and ventilation openings are not covered and do not knock on the casing. After putting the stethoscope to the body, the recording will start automatically. The stethoscope screen and the application will show information on the recording progress. If during recording the stethoscope detects excess ambient noise

or that it is not touching the body in a stable manner, it will signal it with a red icon of the appropriate alarm [see the table in item 3.2 of this manual].


**IMPORTANT!** In the first two cases the system requires the mobile phone to be connected to the Internet.

## 5.1. Patient auscultation mode using a mobile application without connection to the StethoMe AI medical device

It is necessary to launch the application and proceed according to the instructions displayed in the application.

1. Select in the application whether you want to auscultate lungs or heart by touching the appropriate icon.
2. Making a recording consists of putting the stethoscope membrane [Fig. 7] to the patient's body at specific points indicated in the application. The recording is started and finished automatically.
3. The active auscultation point can also be selected without using the application. A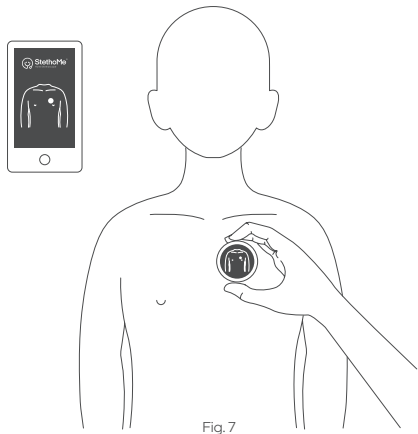

swiping left or right gesture in the stethoscope's horizontal axis is sufficient to select another point.

4. It is recommended to record sound at all the points indicated in the application.

**IMPORTANT! The recording starts automatically once the stethoscope detects that it has been put to the body correctly.**

## 5.2. Patient auscultation mode using a mobile application with connection to the StethoMe AI medical device

It is necessary to launch the application and proceed according to the instructions displayed in the application.

1. Select in the application whether you want to auscultate lungs or heart by choosing the right icon.
2. Making a recording consists of putting the stethoscope membrane [Fig. 7] to the patient's body at specific points indicated in the application and recording sound. The recording is started and finished automatically.
3. The auscultation point can also be selected without using the application. A swiping left

or right gesture in the stethoscope's horizontal axis is sufficient to select another point.

4. It is recommended to record sound at all the points indicated in the application.
5. If the application shows that the quality of sound recorded at some points is poor, it is necessary to repeat recording at those points.
6. After recording all the points properly or after forcing the examination to end with not all of the points recorded, the recording is sent automatically via mobile data transmission for analysis by StethoMe AI algorithms. After the analysis is completed, it is sent back to the user [depending on transmission speed this takes from a few [to several dozen seconds]]. At the same time, the results can also be sent to a medical teleplatform.

**IMPORTANT! The recording starts automatically once the stethoscope detects that it has been put to the body correctly.**

Below you can find the markings employed in the results of StethoMe AI's analysis of the auscultation sounds recorded:

Points on the silhouette:

- green colour - no abnormal auscultation sounds have been detected
- orange colour - a small number of abnormal auscultation sounds have been detected

- red colour - abnormal auscultation sounds have been detected
- red colour with exclamation mark - ERROR - poor signal quality **(No breathing detected or too much interference)**

Status of the whole examination:

- Green lung symbol - no abnormal auscultation sounds have been detected,
- Orange lung symbol - a small number of abnormal auscultation sounds have been detected,
- Red lung symbol - abnormal auscultation sounds have been detected,
- "X" mark - It is impossible to specify the examination result.

**CAUTION!** After obtaining in the application the results of analysis carried out by the StethoMe AI medical device, such results always have to be consulted with a physician.

Check newest features at the address [www.StethoMe.com](http://www.StethoMe.com). You can learn how to read the results from the StethoMe AI user manual, the most recent version of which is available at the address [www.StethoMe.com/manuals](http://www.StethoMe.com/manuals).

### 5.3. Patient auscultation mode without using a mobile application

In order to use the StethoMe stethoscope without the dedicated application, it is enough to connect headphones to the 3.5 mm mini jack socket – the stethoscope screen should then display a lung symbol indicating that the lung auscultation filter is active. After tapping the StethoMe stethoscope screen twice, the filter will be switched to the heart auscultation mode, which will be indicated by the heart symbol on the stethoscope screen.

**IMPORTANT!** Although it is possible to use various type of headphones, due to range of frequencies present in auscultation signals and levels thereof it is recommended to use headphones with minimum impedance of 16 Ohm and bandwidth of at least 20 - 20 000 Hz.

### 5.4. Turning off the stethoscope

After not having been used the stethoscope will enter standby mode. It is not necessary to turn the stethoscope off after each use. In the event of the necessity to turn

the stethoscope off completely, it is possible to do it by tapping the device screen six times. In order to switch the stethoscope on, connect the 3.5 mm jack cable to the socket.

**IMPORTANT! Automatic transition into standby mode (in case of not detecting movement): after about 30 s in case of lack of connection to the mobile application; after about 120 s when the device is connected to the mobile application, but the application is working in the background; after about 5min when the device is connected to the application.**

## 6. Maintenance and cleaning

The stethoscope should be disinfected after each use. It should be disinfected using a cotton pad or swab moistened with non-alcoholic disinfectant intended for medical devices.

- Please take care not to flood the 3.5 mm jack socket with the disinfectant.
- Please take care not to flood the ventilation openings with the disinfectant.
- Do not immerse the stethoscope in water or other cleaning liquids and do not

clean it under running water.

- Make sure that the water does not reach the inside of the device.

## 7. Troubleshooting

You can find a list of solutions for the most frequently occurring issues at the address [www.StethoMe.com/FAQ](http://www.StethoMe.com/FAQ).

## 8. Technical specification

Model:	StethoMe stethoscope
Dimensions:	height - 40 mm, base radius - 30 mm
Display:	TFT-LCD 240x 240 pixels
Equipment:	stethoscope, USB-mini - 3.5 mm jack cable, user manual, quick start guide, warranty card
Power supply:	lithium-ion battery (non-removable)

Battery life:	300 charging cycles or 5 years
Charging voltage/ current:	5V DC / 250mA
Frequency range:	RF: ISM, 2.400-2.4835 GHz
Modulation:	DSSS
Effective radiation strength:	+4dBm

## 8.1. Electromagnetic compatibility and compliance with standards

The device meets the requirements of the IEC60601-1-2 and IEC60601-1 International Standards.

The stethoscope complies with European regulations and features the CE 2274 mark. The quality of the medical device has been verified and complies with provisions of the Council Directive [EC] 93/42/EEC on medical devices of 14th of June, 1993.

## 8.2. Storage, transport and operation conditions

### Storage and transport conditions:

Temperature:	From 0°C to 50°C
Relative humidity:	<85%
Atmospheric pressure:	700 hPa -1060 hPa

### Operation conditions:

Temperature:	From 15°C to 40°C
Relative humidity:	<85%
Atmospheric pressure:	700 hPa -1060 hPa

## 9. Maintenance service

In case of noticing any irregularities in functioning of the medical device, contact the maintenance service at the e-mail address: [support@StethoMe.com](mailto:support@StethoMe.com)

StethoMe Sp. z o.o. bears no liability and provides no guarantee with regard to damage [including indirect damage] arising due to this user manual not being complied with.

## MANUFACTURER

StethoMe sp. z o.o.  
61-663 Poznań; ul. Winogrody 18a  
Poland  
[www.StethoMe.com](http://www.StethoMe.com)

CE  
2274

Date of issue of the manual: 01.07.2019

Version: 02

If the device is marked with the symbol of a crossed-out waste container, it means that this piece of equipment and the batteries included in it may not be disposed of together with other household waste after expiration of their useful life. The user is obligated to hand them over at a waste electric and electronic equipment collection point. The waste collection points, including local collection points, stores and municipal units, create an appropriate system enabling handing over of such equipment.

## NOTES